## Precautions and Warnings

**Description** Neoss Limited manufactures dental implant components from biocompatible titanium and gold alloys. Accessory components are produced in gold alloy, polymers, titanium and stainless steel.

**Procedures** For detailed information on the specific Neoss Implant System product you are using, please consult the manual, Neoss Implant System Guidelines (10501), Neoss Anatomical Healing Abutment Manual (10867), Neoss ø3.25 Implant Manual (10847) and translated versions.

**Indications** Dental implants from Neoss Limited are used as the means of anchorage between crowns, bridges or dentures and the surrounding bone in the upper or lower jaws. Such prostheses may range from replacement of a single tooth to an entire arch of bridgework and cement or screw retained restorations. The Neoss Implants – Neoss Implant System are also intended for immediate placement and function on single tooth and /or multiple tooth applications recognizing sufficient bone stability and appropriate occlusal loading, to restore chewing function. Multiple tooth applications may be splinted with a bar.

**Contraindications** Insufficient size or numbers of implants to support biomechanical loads or undesirable positioning of implants can lead to mechanical failures including fatigue fracture of implant fixtures, abutments or abutment screws. Such an example is a narrow diameter implant in combination with large angulated abutments in the posterior region. Implant placement and prosthetic design must accommodate individual patient conditions such as bruxism or unfavorable jaw relationships to reduce the risk of overload or fatigue failure, and treatment is contraindicated if adequate accommodation cannot be accomplished. Please consult appropriate surgical and restorative manuals and textbooks for information on treatment planning and medical evaluation. Dental implants need to be selected so that they provide an optimal connection and stability with the surrounding bone. Neoss dental implants are

contraindicated where bone of insufficient quality or quantity is available to produce adequate stability and support. The design and construction of the abutment and prosthesis by the technician should incorporate appropriate retentive features for the prosthesis and should optimize the angulation between the implant fixtures and prosthesis such that applied loads are directed down the long axis of the implant. Failure to achieve this can lead to excessive bending force and fatigue failure of the implant components.

The Neoss Implant System has specific design characteristics for mating implants, abutments and prosthetic components. Combining components that are not configured or dimensioned for correct mating can lead to mechanical failure of components, damage to tissue, or unsatisfactory esthetic results.

Pre-operative hard tissue or soft tissue deficits may result in a compromised esthetic outcome or unfavorable implant angulation.

**General Precautions** Surgical and restorative products used to achieve and maintain osseointegration should be utilized by persons trained in this method. Such training is offered at a number of centers. Please contact the manufacturer for information. Pre-operative patient evaluation and close cooperation between surgeon, restorative dentist and dental laboratory technician is essential for success. Neoss implants, abutments and abutment screws must be used solely on one patient.

**Procedural Precautions** Especially important is proper stress distribution: relining of the denture after implant installation to avoid premature loading; passive adaptation and fitting of the bridge to the implant abutments; adjusting occlusion to the opposing jaw; avoiding excessive transverse loading forces are all important considerations. Prosthesis metal substructures should be made using either titanium or high gold content alloys. Abutments or other components of non-precious metal alloys other than titanium should not contact the implant. Because of the small size of

prosthetic components, care must be taken that they are not swallowed or aspirated by the patient.

**Adverse Effects** Implant techniques have normal contraindications and risks. These are extensively documented in the dental literature. Incorrect clinical placement or loading resulting in loss of implant anchorage or loss of the prosthesis are possible events after surgery. Lack of bone quantity or quality, infections, poor patient hygiene or cooperation, and general diseases are some potential causes for loss of anchorage and function.

**Sterility** All dental implant, bone cutting instruments and some healing abutments are supplied sterile with a given expiry date as indicated by the packaging. For re-sterilization of non-sterile or sterile instruments and accessories please refer to the Neoss Implant System Guidelines (10501).

**Caution:** Federal (USA) law restricts the sale of this device to or on the order of a licensed physician or dentist.

# Instructions for use

Neoss Implant System

Implants, Abutments and associated dental, surgical, technical and restorative components

**Neoss Limited**

Windsor House, Cornwall Road  
Harrrogate HG1 2PW  
United Kingdom

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